Livit

Blood Collection Kit

Instructions

2–7

Käyttöohje

8–13

Instruktioner

14–19

Anweisung

20–25

Juhend

26–31

Nightingale Health

Welcome to the
new era of
preventative
health.

NGH-0302

Nightingale Health Oyj
Mannerheimintie 164a
00300 Helsinki, Finland

Hello!

Congratulations on your decision to take a step towards lifelong health.

Livit by Nightingale Health helps you better understand how your body is doing right now and how to improve your health.

Before you begin, read the instructions from start to finish so that you know what to do. It shouldn't take you more than half an hour to collect the sample. If you feel nervous about taking the blood sample, don't worry – pricking your finger only pinches a little and you just need about 4–5 drops of blood.

FAQ: support.nightingalehealth.com

Customer support: support@nightingalehealth.com

THE TEST PROCEDURE IN BRIEF

01

REGISTER KIT IN APP

02

FAST FOR 10–12 HOURS

03

COLLECT SAMPLE

04

MAIL SAMPLE

05

RECEIVE RESULTS

01


Register kit in app

Start by registering your test kit in the Livit by Nightingale Health app. Registration is necessary for us to deliver your results to you.

If you don't have the app yet, download it from the App Store or Google Play.

Livit by Nightingale Health →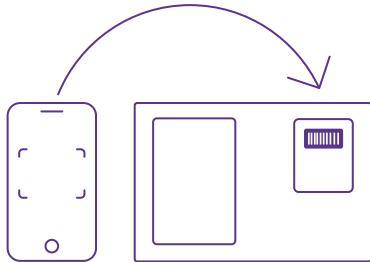

To register your test, use a mobile device to scan the barcode on the back of the Livit Blood Collection Kit package.

After signing in to your account on the app, tap **“Register Kit”** and follow the instructions.

02

Fast for 10–12 hours

You should not eat or drink anything except water for 10–12 hours before collecting your sample. We recommend that you fast overnight and collect your sample in the morning.

Drinking water during your fast is helpful, as it increases your blood volume and makes the sample collection easier.

You can take your usual medications as normal.

KEEP THE DELIVERY BOX

Keep the delivery box, you'll need it to return the sample by mail.



03

Collect sample


Make sure you follow the instructions closely so that your test results will accurately reflect your health.

Start by laying out all of the test items in front of you:

- Velvet Blood Collection Device in a pouch
- Return Package Kit (includes desiccant and oxygen scrub)
- Supplies Set:
 - A Alcohol pads (2 pcs)
 - B Single use lancets (2 pcs)
 - C Bandages (2 pcs)
 - D Gauze pad
- Instructions for use
- Cardboard delivery box (you'll need this to mail your sample)
- Paperboard base → you can use this as a test base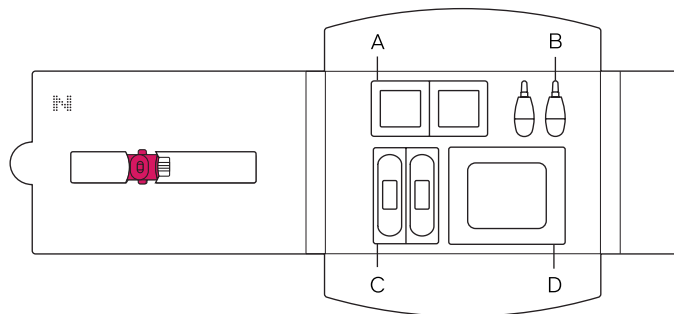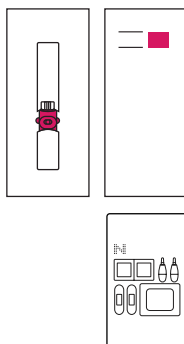


1

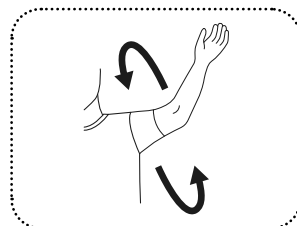Start by warming up. You can windmill your arms, massage your hands and do jumping jacks, for example.

2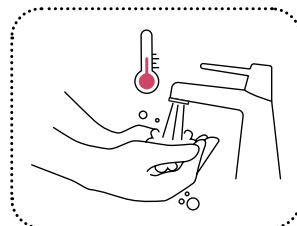


Wash your hands with soap and running warm water for a few minutes. Massage your fingers to warm them up. This improves the circulation of blood in your fingers, which makes it easier to collect the sample.


3

Place the test items in front of you in their indicated positions on the test base and remove the following items from their packaging:

- Blood Collection Device
- Gauze pad
- One bandage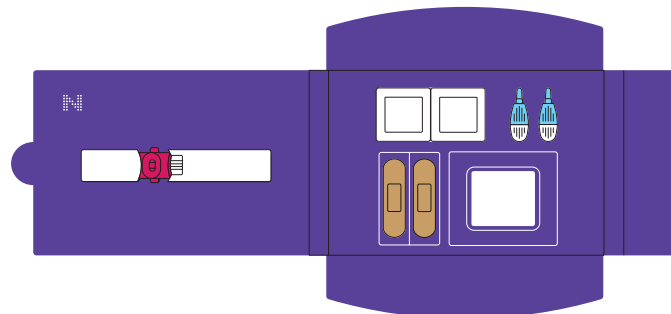


4

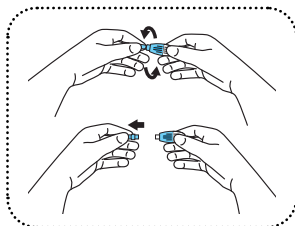
Open one lancet by twisting the thin blue cap and pulling it off.

5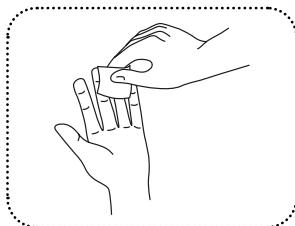


Clean one finger by using an alcohol pad and let it dry. We recommend collecting the sample from the ring finger of your non-dominant hand.

6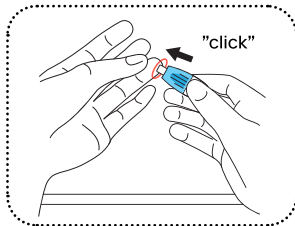


Lay the clean finger on a table. Firmly press the lancet against the side of your fingertip until you hear a click. Wipe away the first drop of blood with gauze.

7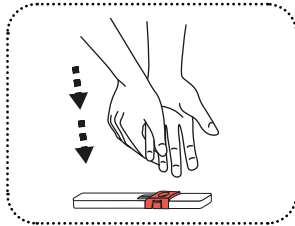


Relax your arm, keep your finger pointed downwards and slowly massage your finger repeatedly from your palm towards the last joint to create blood drops. Don't worry if you don't see a drop of blood forming right away – collecting the sample may take a few minutes.

8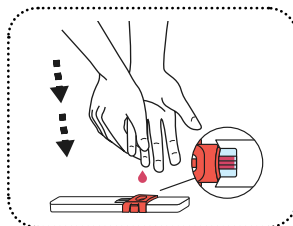

Let drops of blood fall into the red funnel on the device while keeping your arm relaxed. If necessary, you can gently bring the funnel into contact with a drop of blood to help it fall into the funnel.


9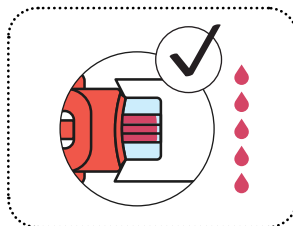

You've collected enough blood when all three tubes are full. This usually takes about 4–5 drops of blood. You can see the tubes filling through the window on the device.


Sometimes the blood flow can stop too early. If this happens, you can repeat the procedure using the middle finger of your non-dominant hand and the other lancet in the box. Try to continue the sample collection until all three tubes are full. If the amount of blood is insufficient, we might be unable to deliver your results.

10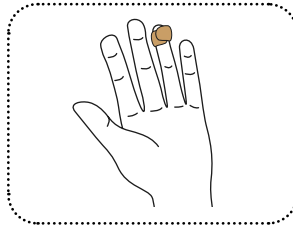


Wipe the finger with gauze, apply pressure where the finger was pricked to stem the flow of blood, and then put on a bandage.

11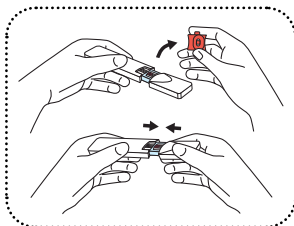


Remove the red funnel from the device. Close the device by pressing it slowly from both ends until you hear a click. It's normal that blood spreads around inside the device.

12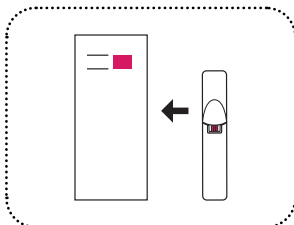


Open the Return Package Kit by tearing it at the notch and place the device in the bag. Don't remove anything from the bag and seal the bag carefully.

13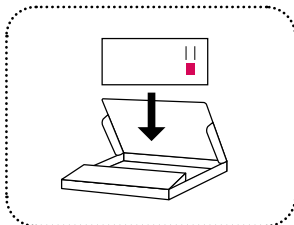

Place the Return Package Kit in the cardboard delivery box and use the adhesive tape on the inner lid to close the box.

14

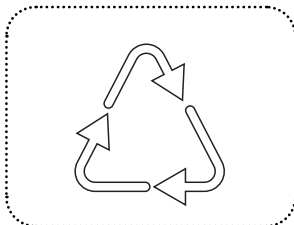Collect the lancets and other used sample collection instruments, place them in the blood collection device bag, and dispose of them in mixed waste. The instructions for use can be recycled as paper waste. Dispose of the remaining items in accordance with your country's recycling guidelines.

04

Mail the sample within 12 hours

First, make sure you've registered your kit so that we can link your results to your account and deliver them to you. Then, mail the sample within 12 hours of collecting it. Follow the mailing instructions closely to ensure the stability of the sample.

Refer to the back of the delivery box for more detailed country-specific mailing instructions.

05

Receive results

We will send you a message as soon as our laboratory has analyzed your sample and your results are ready. You will see your results in the Livit by Nightingale Health app.

General instructions

Intended use

Livit Blood Collection Kit is intended for the collection, separation of plasma, storage, and transport of a whole blood sample from a fingerstick for in vitro diagnostics purposes. The blood sample can be self-collected by an adult lay user at home or can be collected by a healthcare professional. The device is for single use only.

Warnings and precautions

Do not use if the user has a bleeding disorder.

Do not use after the expiration date indicated on the box.

Do not use if the package is damaged.

For single use only.

For in vitro diagnostic use only.

Keep out of reach of children.

For external use only.

Minor bruising may occur around the sample collection site.

Intended for self-use or use by a healthcare professional.

Use while seated, as fainting may occur with any blood sampling procedure.

In case any serious incident occurs in relation to the device, it shall be reported to the manufacturer and the competent authority of the country in which the incident occurred.

Storage and stability

The device should be stored at room temperature 15–30°C (59–86°F) and it can be used until the expiration date. The sample is to be mailed at ambient temperature by using the materials provided for mailing.

CE mark

In vitro diagnostic medical device

Manufacturer

Consult instructions for use

Do not re-use

Do not use if package is damaged

Temperature limit

Use-by date

Catalogue number

Batch code

Serial number

Biological risks

Authorized representative in the
European Community/European Union

Does not contain natural rubber latex

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FAQ: support.nightingalehealth.com/fi

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