# Instructions for Use

---

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

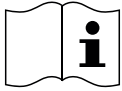
## Important information – Read before proceeding with sample collection

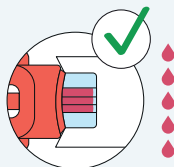
1. Ensure that you collect the required amount of blood. All three tubes must be completely filled.

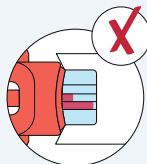If the blood volume is too low, we will not be able to provide test results. Use the second lancet if necessary.

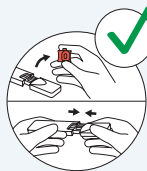2. After filling all three tubes, remove the red funnel from the device and close the device by pressing both ends together.

3. Place the device in the return bag. Do not remove anything from the bag and seal the bag carefully.

For optimal test results, avoid eating and drinking caffeinated beverages – such as coffee or tea – for at least 3 hours before collecting the sample.

## Sample Collection Steps

### Preparing for Collection:

#### The most important step:

Wash your hands under warm running water for 3–5 minutes. This is the single most effective way to increase capillary blood flow to your fingertips and ensure successful sample collection.

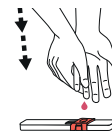
Rotate your arms in large circles and massage your hands to further improve blood circulation in your fingers. Stand upright when collecting the sample, as this helps blood flow down to your fingertips.

### Collecting the Sample:

Clean one finger with the alcohol pad and allow it to dry. Open the lancet, place your hand on the table, and press the lancet against the side of your fingertip until you hear a click and the lancet pricks the finger. Wipe away the first drop of blood with gauze pad.

Relax your arm, keep your finger pointing downwards, and gently massage your finger repeatedly from the palm toward the fingertip to encourage blood flow. Allow the drops of blood to fall into the red funnel on the device.


You have collected enough blood when all three tubes are completely filled. If you cannot fill all three tubes, repeat the process using another finger and the second lancet.

### After sample collection
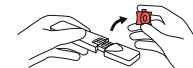
#### Closing the device:


After all three tubes are completely filled, remove the red funnel from the device. Close the device by pressing both ends together until you hear a click.

#### Returning the device:

Place the device in the return bag. Do not remove anything from the bag and seal the bag carefully. Place the return bag in the cardboard delivery box and use the tape to close the box. **Send the sample within the same day of collection, using the return instructions provided to you.**
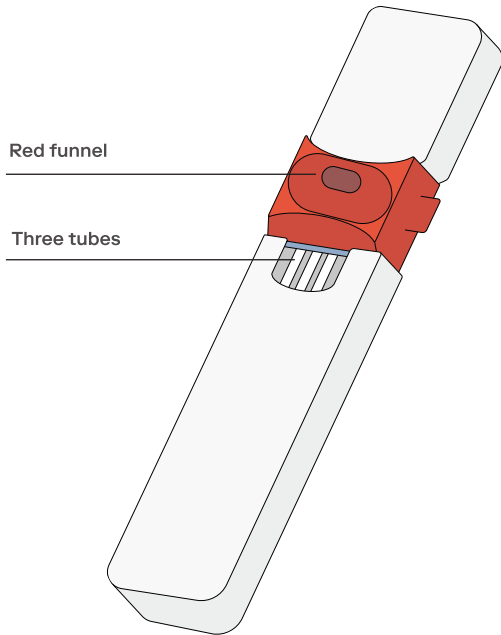
## Blood Collection Device

Red funnel

Three tubes

### Contents:

Blood Collection Device, single use lancets, alcohol pads, gauze pad, adhesive bandages, Blood Collection Device Return Bag.

### General instructions

#### Intended use

Livit Blood Collection Kit is intended for the collection, separation of plasma, storage, and transport of a whole blood sample from a fingerstick for in vitro diagnostics purposes. The blood sample can be self-collected by an adult lay user at home or can be collected by a healthcare professional. The device is for single use only.

#### Warnings and precautions

Do not use if the user has a bleeding disorder.

Do not use after the expiration date indicated on the box.

Do not use if the package is damaged.

For single use only.

For in vitro diagnostic use only.

Keep out of reach of children.

For external use only.

Minor bruising may occur around the sample collection site. Intended for self-use or use by a healthcare professional. In case any serious incident occurs in relation to the device, it shall be reported to the manufacturer and the competent authority of the country in which the incident occurred.

#### Storage and stability

The device should be stored at room temperature 15–30°C (59–86°F) and it can be used until the expiration date. The sample is to be returned at ambient temperature by using the materials provided for sending.

	CE mark		Serial number
	In vitro diagnostic medical device		Biological risks
	Manufacturer		Authorized representative in the European Community/European Union
	Unique Device Identifier		Does not contain natural rubber latex
	Consult instructions for use		Keep dry
	Do not re-use		Keep away from sunlight
	Do not use if package is damaged		Sterilized using ethylene oxide
	Temperature limit		Sterilized using irradiation
	Use-by date		Caution (sharps)
	Catalogue number		Date of manufacture
	Batch code		Medical device

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